Breath -A- Tech® Cardboard Spacer

- Anti-static
- No priming required before use
- Compact and portable
- Easy expansion
- Can be used with most pressurised metered dose inhalers
- Single patient use, for up to seven days
- Australian made
- Remove exhalation valve before recycling

Medical Developments International Limited ABN 14 106 340 667
4 Caribbean Drive, Scoresby Victoria 3179, Australia
Telephone +61 3 9547 1888 Fax +61 3 9547 0262
Email mdi@medicaldev.com Web www.medicaldev.com

REF BAT-CARD

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

8024MDI LI-BAT-CARD-V1

Indications for Use

ENGLISH:

The Breath-A-Tech® Cardboard Spacer is a compact and portable collapsible spacer intended to be used by patients to administer aerosolised medication from most pressurised metered dose inhalers prescribed by a physician or healthcare professional. The intended environments for use include home, hospitals, emergency services and clinics.

The Breath-A-Tech® Cardboard Spacer does not need priming before use and can be used straight out of the package.

Please note

- Always follow the instructions provided with your inhaler.
- Before use, check that the Breath-A-Tech® Cardboard Spacer is intact and that the interior is free of small objects that could be inhaled.
- Replace the Breath-A-Tech® Cardboard Spacer immediately if it is damaged.
- Store in a dry area away from direct sunlight.
- Breath-A-Tech® Cardboard Spacer can be used multiple times. However, it is recommended to replace the Breath-A-Tech® Cardboard Spacer after 7 days or sooner if it becomes damaged, soggy or dirty.

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